
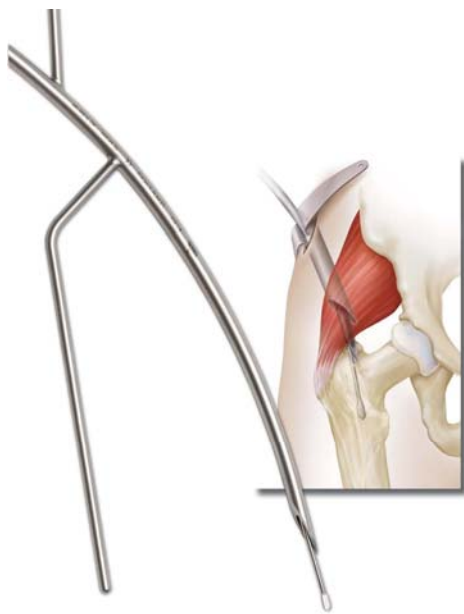


Femur Finder System

Helps assure accurate guide wire placement during femoral IM fixation, while minimizing soft tissue trauma

Product No's:
6001 [Femur Finder System]
System Includes:
6005 [Femur Finder]
6010 [Tissue Protector]
6015 [Sterile Guide Wires Spade Tip - 5PK]
6016 [Sterile Guide Wires Ball Tip - 5PK]
6018 [Cleaning Brush]
6020 [Sterilization Case]

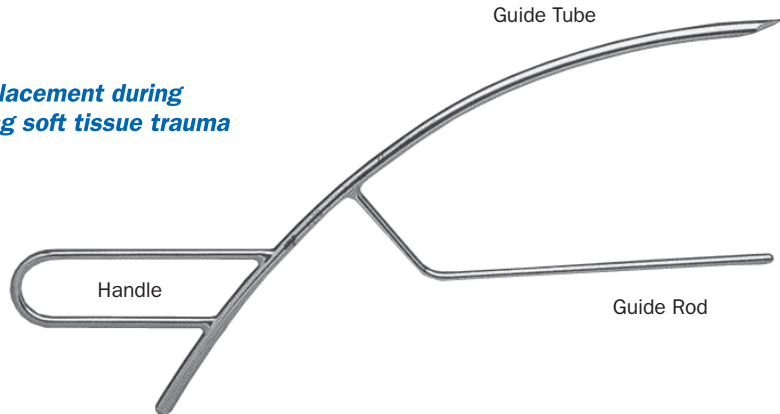

Eliminates the need to "eyeball" the correct femoral entry point. The instrument facilitates aiming, drilling and directing of guide wire accurately into the medullary canal during femoral nailing. The uniquely designed Spade Tip guide wire, which works with any femoral nail instrumentation, prevents debris from forming in the wound area during drilling.

Both the Spade Tip and the Ball Tip guide wires are marked for proper depth of penetration and chuck location. The Tissue Protector allows reamers of up to 15 mm in diameter to pass through the incision with minimal soft-tissue damage and muscle dissection. Its flexible wings can be bent to conform to the radius of the thigh, and rings in the end of wings facilitate clamping the Tissue Protector to the drapes. This frees the assistant from holding the device and keeps hands out of the path of the image intensifier beam. The complete system includes a custom fitted sterilization tray.

FREE TRIAL
ON MOST INSTRUMENTS
CALL FOR MORE INFORMATION



©2006 Innomed, Inc., All Rights Reserved

Phone 800 548 2362
912 236 0000
Fax 912 236 7766
Internet www.innomed.net
eMail info@innomed.net

Innomed
ORTHOPEDIC INSTRUMENTS
103 Estus Drive Savannah GA 31404

