

1-800-548-2362

International (912) 236-0000



www.innomed.net

(912) 236-7766 Fax

©2000 Innomed, Inc., All Rights Reserved.

Innomed

Instrument Division
103 Estus Drive, Savannah, GA 31404



FREE TRIAL ON MOST INSTRUMENTS
CALL FOR MORE INFORMATION

Fracture Alignment Device

Used to align the bone ends of a femur fracture. This allows the passing of a guide wire across the fracture site. After the proximal portion of the femur is reamed over a guide wire, the Fracture Alignment Device is inserted and used to help align the bone ends. A guide wire can then be passed through the Fracture Alignment Device and across the fracture site. The Fracture Alignment Device is then removed leaving the guide wire across the fracture site.

Designed by
R. Audell, M.D.

Product No: 8310

*Used to align the
bone ends of a femur fracture*

