

# Universal Modular Femoral Hip Component Extractor

*Helps remove a femoral hip stem after the modular head has been removed*

Designed to clamp onto the taper of a femoral hip stem after the modular head has been removed. The extractor is equipped with a swivel block for attachment of a slap hammer. The swivel block helps keep the slap hammer in line with the angle of the femoral stem. Includes standard slap hammer, #3925.

#### PRODUCT NO'S:


**3610** [Extractor with Standard Slap Hammer #3925]

#### Optional/Individual Parts:


**3610-01** [Extractor Only]

**3925** [Standard Slap Hammer] 3/8"-16 Thread Gauge


**3935** [Extra Large Slap Hammer] 3/8"-16 Thread Gauge

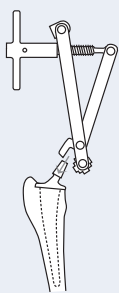


The extractor is opened to accommodate any size taper on a modular head total hip stem.

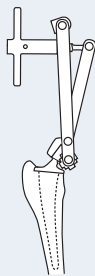

The taper is clamped between the rotating block and the taper anvil. Tightening the "T" handle holds a stem taper in place.

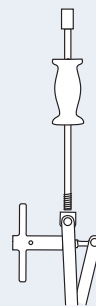

The slap hammer is screwed into the swivel block. The slap hammer can be aligned with the stem utilizing the swivel block.

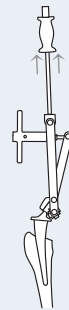

Extraction is carried out by the slap hammer or by utilizing a mallet on the hammer flares of the slap hammer.

