


Lombardi Femoral/Gluteus Medius Minimus Retractor

Designed by Adolph V. Lombardi Jr., MD

Designed for acetabular exposure, and to retract the gluteus medius minimus during femoral reaming

For acetabular exposure, the retractor can be placed easily at the level of the ischium and driven into the ischium to retract the femur posteriorly when using an anterolateral approach to the hip. Also using an anterolateral approach or a modified Harding hip approach, the retractor can be placed in the tip of the greater trochanter and can effectively retract the abductor mechanism, namely the gluteus medius minimus so that reaming of the femur can be performed.

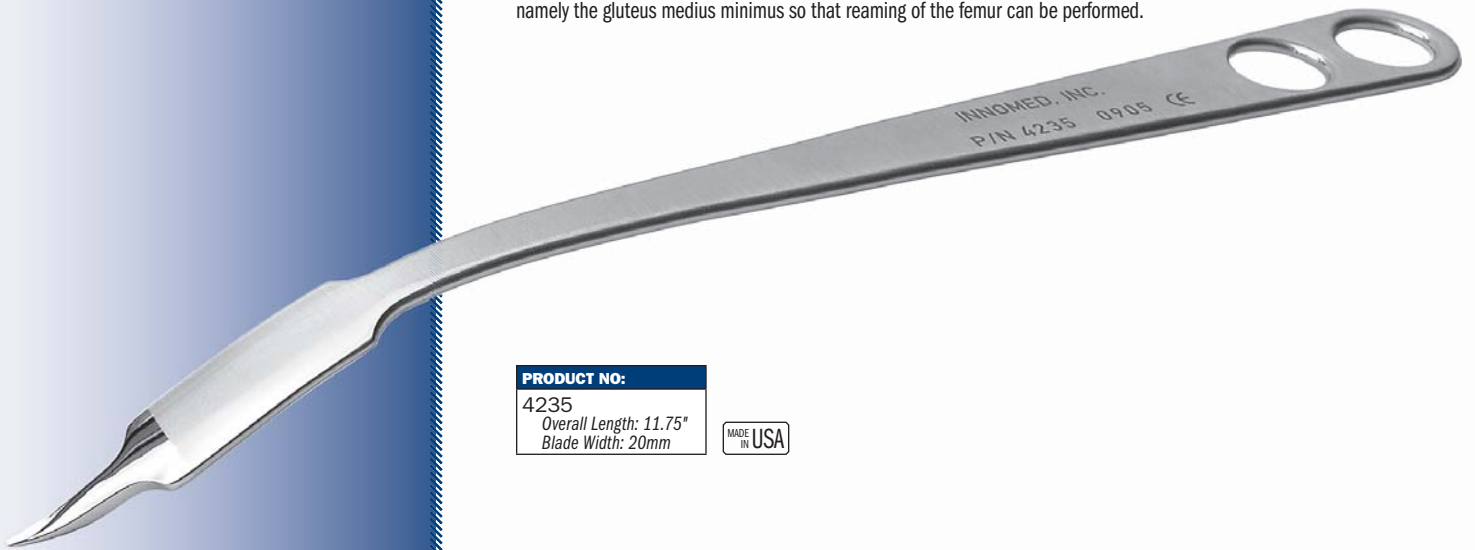

PRODUCT NO:

4235

Overall Length: 11.75"
Blade Width: 20mm

MADE
IN USA

FREE TRIAL ON MOST INSTRUMENTS

© 2011 Innomed, Inc.

INNOMED



103 Estus Drive, Savannah, GA 31404
www.innomed.net info@innomed.net

912.236.0000 Phone
912.236.7766 Fax

Innomed-Europe Tel. +41 41 740 67 73
Fax +41 41 740 67 74

1.800.548.2362