


Meftah PCL Protector

Designed by Morteza Meftah, MD

Designed to help protect the posterior cruciate ligament in cruciate retaining total knee surgery during the proximal tibial cut

The PCL Protector can be used efficiently right before the tibial cut. It is curved distally so that it can put over the PCL from the top/posterior side and with a few taps, the fanned blade can get around the PCL and into the bone (not more than 5 mm) and "cover" the PCL. The protector is left in place until the tibial cut is made with a saw, which would hit the protector instead of the PCL if it gets too close.

PRODUCT NO:
3221
Overall Length: 8" (20,3 cm)

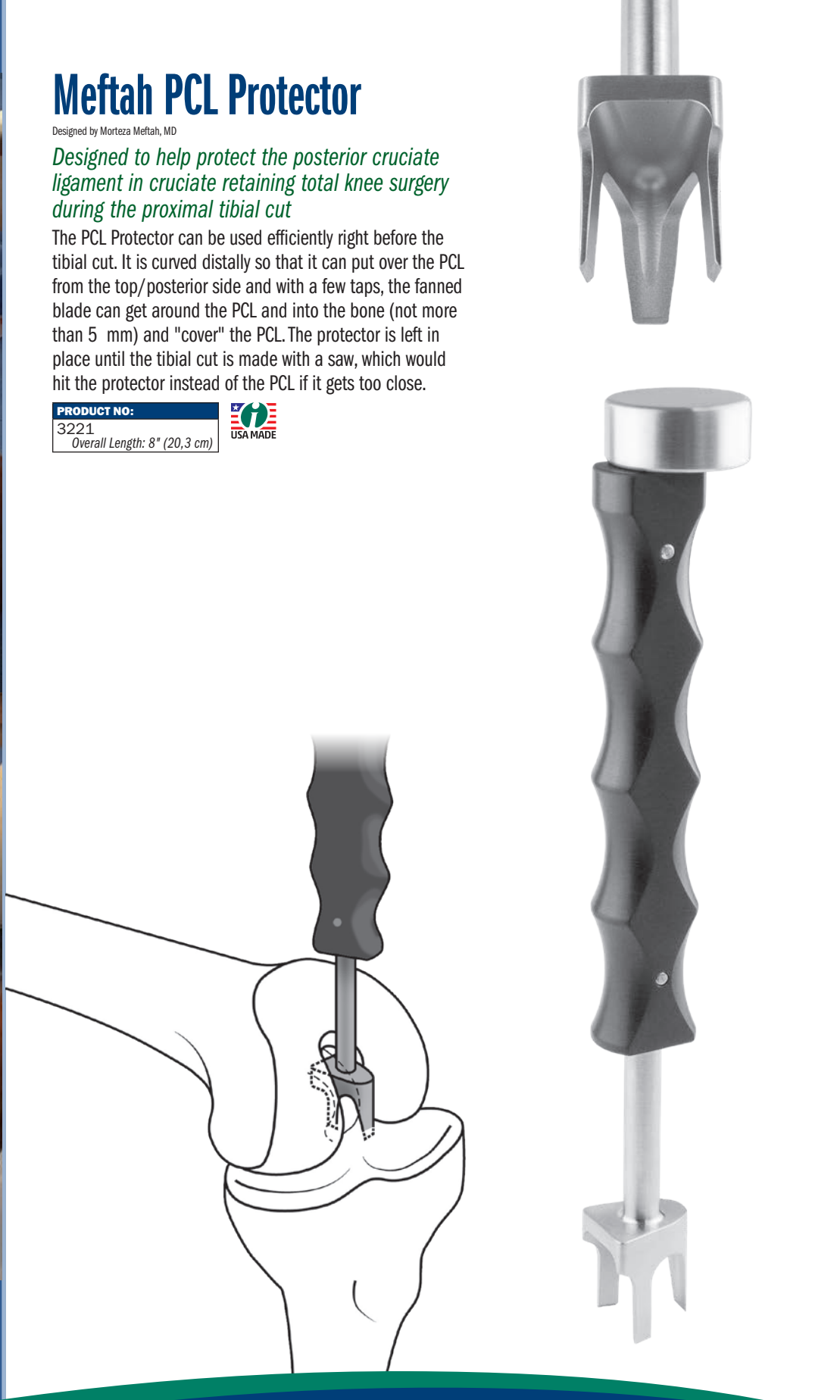

www.innomed.net

ISO 13485:2016

FREE TRIAL ON MOST INSTRUMENTS

INNOMED 



103 Estus Drive, Savannah, GA 31404
info@innomed.net

912.236.0000 Phone
912.236.7766 Fax

Innomed-Europe Tel. +41 41 740 67 74
info@innomed-europe.com

1.800.548.2362