

Screw Extractor Set

Designed to help remove screws with stripped or damaged heads


- ▶ Extractors must be used with drill in reverse.
- ▶ Screw head is reamed with burnishing end, and is then removed with the left turn thread end.
- ▶ Care must be taken to burnish no more than 1/16" (1.5 mm) deep, as burnishing too deep can weaken the screw head.



Burnishing Ends



Left Turn Thread Ends

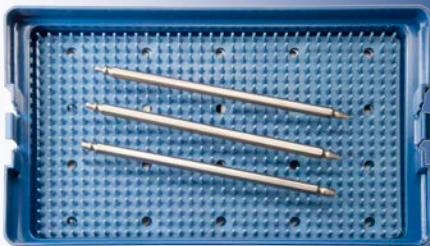


PRODUCT NO:

7250-00	[Set with Case]
7250-01	[2.5 mm Screw Extractor] Overall Length: 6" (15,2 cm)
7250-02	[3.5 mm Screw Extractor] Overall Length: 6" (15,2 cm)
7250-03	[6.5 mm Screw Extractor] Overall Length: 6" (15,2 cm)



USA MADE

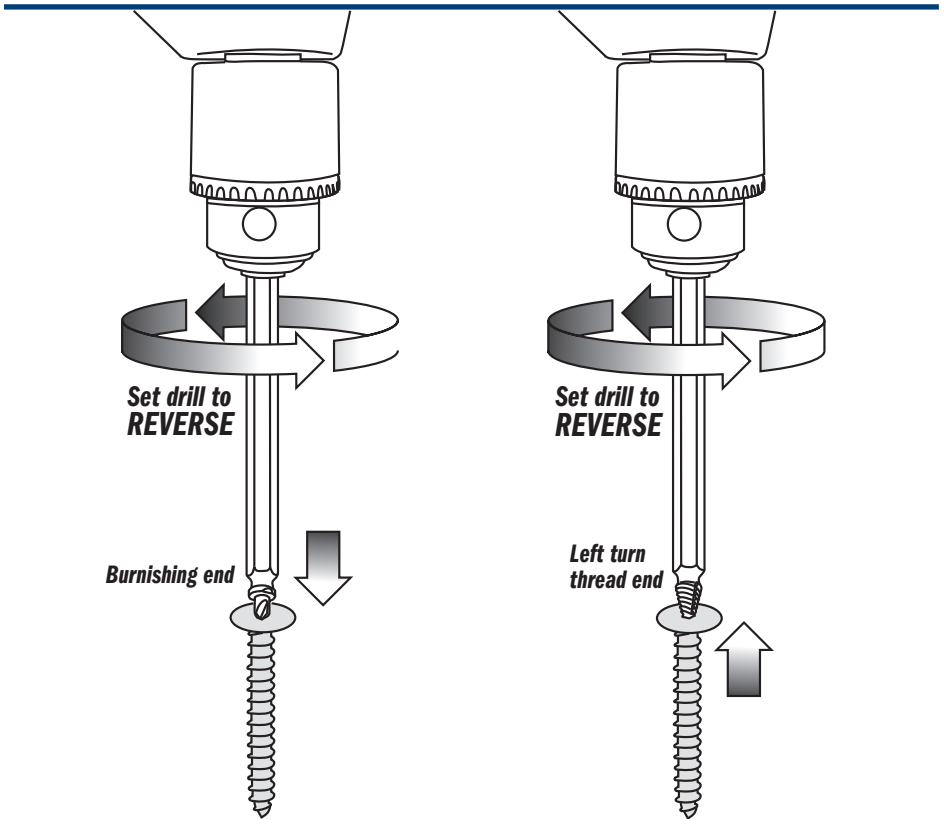

ISO 9001:2008 • ISO 13485:2003

FREE TRIAL ON MOST INSTRUMENTS

Scan to
Launch Our
Website



103 Estus Drive, Savannah, GA 31404
www.innomed.net info@innomed.net

912.236.0000 Phone
912.236.7766 Fax

Innomed-Europe Tel. +41 41 740 67 74
Fax +41 41 740 67 71

1.800.548.2362

© 2016 Innomed, Inc.