




Adjusting Blades To Fit Component

The straight or angled blades are adjusted by loosening the attached screws and sliding the blades into the desired position.

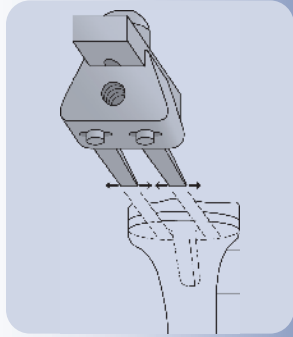



Driving Blades Under Component

The blades are driven under the tibial base.

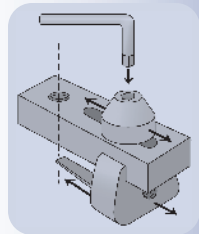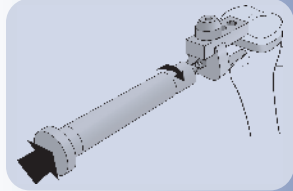

Tightening Threaded Rod Onto Component

The site hole for the pointed, threaded rod can be aligned with the proximal surface of the tibial component by using the included hex wrench system. The pointed, threaded rod is tightened onto either a polyethylene or metal tibial component.

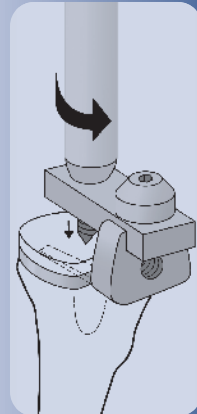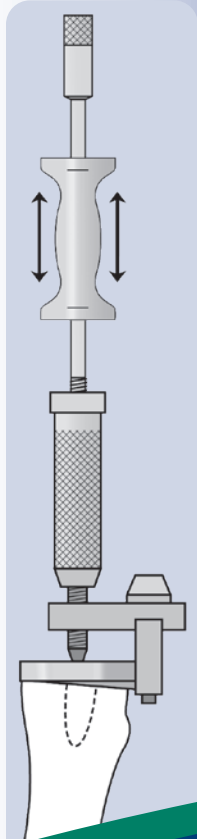


Attaching Slap Hammer Assembly & Removing Component

The slap hammer assembly is threaded into the threaded rod handle for removal of the component.

Tibial Component Extractor

Universal extraction instrument for total knee revision surgery

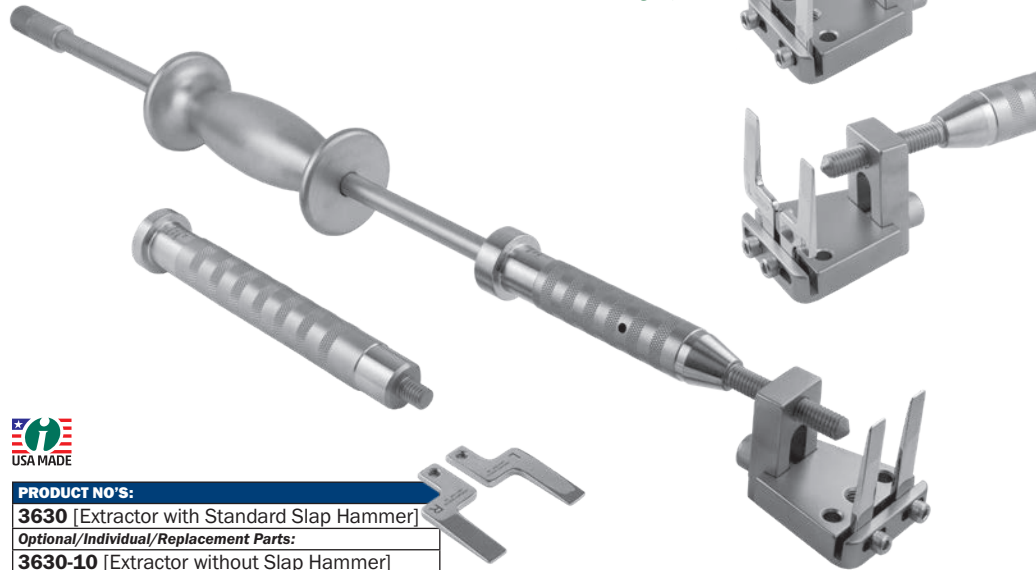

PRODUCT NO'S:

3630 [Extractor with Standard Slap Hammer]

Optional/Individual/Replacement Parts:

3630-10 [Extractor without Slap Hammer]

3630-01 [Pair of Standard Blades]
10mm x 50mm

3630-02 [Pair of Offset Blades]
10mm x 50mm, Offset 15mm

3630-HS [Hex Screws] Pkg of 6


3925 [Standard Slap Hammer]
Thread Gauge: 3/8"-16

3935 [Extra Large Slap Hammer]
Thread Gauge: 3/8"-16



Clamps onto a tibial knee component for extraction

The Tibial Component Extractor is designed to lock onto a tibial component and extract in line with the stem or pegs. Two adjustable osteotomes are inserted on the underside of the component. A locking screw clamps on to the top of the extractor to secure the component. Includes standard slap hammer #3925.

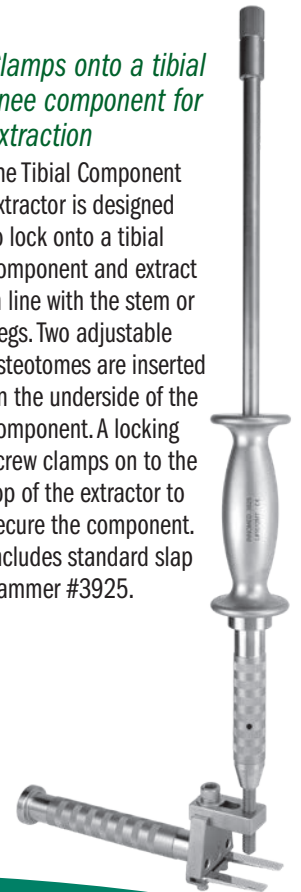

www.innomed.net



FREE TRIAL ON MOST INSTRUMENTS

INNOMED

ISO 13485:2016



103 Estus Drive, Savannah, GA 31404
info@innomed.net

912.236.0000 Phone
912.236.7766 Fax

Innomed-Europe Tel. +41 41 740 67 74
info@innomed-europe.com

1.800.548.2362

© 2024 Innomed, Inc.