



Trans-sulcus Angle Guide

Designed by Richard Scott, MD

Helps establish the trans-sulcus line

A line is drawn down the deepest part of the trochlear sulcus (Whiteside line) with a marking pen or cautery. The post on the guide is inserted into the hole in the femur made for an intra-medullary alignment guide. The trans-sulcus angle guide is then rotated until the line on the guide lines up with the Whiteside line. A line is then drawn along the bottom of the guide.

PRODUCT NO:

1160

Dimensions: 2.25" x .75" (5,7 cm x 1,9 cm)

Post Depth: 1.5" (3,8 cm)



www.innomed.net

ISO 13485:2016

FREE TRIAL ON MOST INSTRUMENTS

INNOMED



103 Estus Drive, Savannah, GA 31404
info@innomed.net

912.236.0000 Phone
912.236.7766 Fax

Innomed-Europe Tel. +41 41 740 67 74
info@innomed-europe.com

1.800.548.2362