

1-800-548-2362

International (912) 236-0000



www.innomed.net

(912) 236-7766 Fax

©2004 Innomed, Inc., All Rights Reserved.

**Innomed**

103 Estus Drive, Savannah, GA 31404  
E-Mail: info@innomed.net



FREE TRIAL ON MOST INSTRUMENTS  
CALL FOR MORE INFORMATION

## Incision Aligner

The bent ends of the aligner are placed at each end of an incision, which is aligned by pulling outward on each end. The sliding end will lock in place when it is tensioned. Pressing inward slightly on the sliding end will allow the aligner to be collapsed and removed.

Designed by  
DMP

Product No: 1330

*Designed to align an incision during closing*

